# Cleaning & Drying UV DRY BOX

**Bedienungsanleitung**  
**Instruction Manual**  
**Manuel d'utilisation**  
**Manual de instrucciones**  
**Manuale di istruzioni**  
**Gebruiksaanwijzing**  
**Brugervejledning**  
**Instrukcja obsługi**

**HADEO**  
*clean & care by HANSATON*

### **DE** Warnhinweis:

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### **EN** Warning:

This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning use of the appliance in a safe way and understand the hazards involved. Children shall not play with the appliance. Cleaning and user maintenance shall not be made by children without supervision.

This dry box shall be only supplied by the provided adapter. Don't use the device in a wet or damp environment.

These instructions are also available in an alternative format.

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## Instruction Manual

The Clean and Drybox cleans the hearing aid by means of UVC and dries by means of heating. Hereby the moisture is removed by the heating cycle. The UVC light removes dirt and the possible presence of bacteria from your hearing aids. This allows the hearing aid to perform better, so that it creates a better hearing effect and can help to prevent skin irritations, itching, rashes or ear infections.

### WARNING:

The Clean & Dry Box 3.2 CONTAINS HIGH PRESSURE. IT SHOULD ONLY BE OPENED OR REPAIRED BY PROFESSIONALS.

SUITABLE FOR RECHARGEABLE HEARING EQUIPMENT

IMMEDIATELY REPORT ANY SERIOUS INCIDENTS TO THE MANUFACTURER.

### Instructions for use:

1. Please insert one end of the supplied USB-C cable into the USB-C port of the Clean & Dry Box 3.2.
2. Insert the other end of the cable into the USB port of the power source. After connecting it to the power grid, five LEDs on the Clean & Dry Box 3.2 will light up from left to right. This will be repeated three times, at which point the Dry Box is now ready for use. The standby function is now enabled.
3. Clean the hearing aid according to the manufacturer's cleaning instructions. After wiping it with a dry towel, place the hearing aid inside the Clean & Dry Box 3.2, distributed across the right and left side of the tray and close the lid. For a non-rechargeable hearing aid, remove the batteries from the device.
4. Press the start button once. The five LEDs will light up (four white and one blue) and the Clean & Dry Box 3.2 will begin its 5-hour cycle.
5. When all of the LEDs are turned on, the warm air flow starts to work. An airflow of 30 degrees Celsius flows along the sides of the box to warm up the hearing aid during this period. Simultaneously, the UVC light (wavelength 254 nm) is turned on, which cleans the hearing aid. After approximately

eight minutes, the cleaning process is complete and the UV-C light turns off, however the flow of hot air continues.

The LEDs indicate the remaining heating period:

First 75 minutes	LED 1 blinks and then turns off
Second 75 minutes	LED 2 blinks and then turns off
Third 75 minutes	LED 3 blinks and then turns off
Fourth 75 minutes	LED 4 blinks and then turns off

Finally, LED 5 will turn green and start to blink. The entire 5-hour cycle is now complete. The flow of hot air will be switched off. If the device is accidentally opened while the UV-C light is still on, the computer will turn off the UV-C light safely and end the calculated UV-C time. After completion of the heating period, the Clean & Dry Box 3.2 will turn off automatically and the green LED will continue to blink for 3 hours. If you have to interrupt the dehumidification process or the UV-C process, press and hold the start button for at least 3 seconds or disconnect the power source.

6. Let your hearing aid cool down for at least 5 minutes, then the device is ready for use again.
7. When using a transformer, the delivered Voltage should be between 4.75V and 5.25V, and the current should be approximately 500mA, otherwise the Clean & Dry Box 3.2 may break down or it may not work properly.
8. The Clean & Dry Box 3.2 automatically recognizes the USB2.0 computer current (500mA); you only need to connect it to the USB port. If the USB power source only supplies 500mA, the Clean & Dry Box 3.2 will be set to 500mA mode. Because the heating power is small, the temperature might not reach 30 degrees Celsius. If the USB plug is not inserted correctly or the effectively supplied current is below 500mA, the Clean & Dry Box 3.2 will stop working. When you want to leave the 500mA mode, please disconnect the transformer for at least 10 seconds. Reconnect the device with a transformer that supplies a current of more than 1A, and a Voltage of

5V. The Clean & Dry Box 3.2 will restart in 1000mA mode.

9. When the device is not in use for a longer period of time, or when leaving the house, please disconnect the device from the power source.

**Care of the UVC Clean & Drybox 3.2:** When cleaning your UVC Clean & Dry Box 3.2, first disconnect it from the power supply. We recommend using moist wipes for cleaning the drying station. Please avoid direct contact with water. Only the inner element can be removed and may be rinsed under running water (see illustration on page 4).

### Specifications:

Model	TC-CDB-20
Input	4.75-5.25V
UVC light	1.5W +-10%
Heating	3.5W +-10%
Heating time	5 hours
UVC time	8 minutes
Adaptor input	100 V- 240 V;50 HZ / 60 HZ; 0.2 A
Output	5 V d.c.; 1000 mA
Working conditions	Indoor use
Operation altitude range	0 m up to 2000 m
Operating temperature	5 degree Celsius to 40 degree Celsius
Supply voltage fluctuations	Not more than 10 %
Pollution degree	Class II

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**WAKE UP INTERNATIONAL B.V.**