

Inserting and Changing the Battery.



Low Battery Warning

Two long beeps indicate the hearing system battery is low. After the low battery warning, sounds may not be as clear. This is normal and can be fixed by changing the batteries in the hearing systems. If you are not able to hear the low battery warning, your hearing healthcare professional can change its pitch or loudness. If you prefer, it can be turned off entirely.

- Open the battery door.
- Insert the battery into the battery compartment, so that the negative pole faces into the door.
- Close the battery door

FCC ID:

VMY-UWITE2

VMY-UWITE3

VMY-UWITE4

VMY-UWITE1 VMY-UWITE4 IC ID:

2756A-UWITE2

2756A-UWITE3


2756A-UWITE4 2756A-UWITE1

2756A-UWITE4



With a new battery, it may take a few minutes for the battery to develop its full output. Only then should you close the battery compartment.

Inserting and Removing the Hearing System.

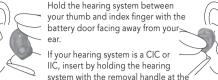


 If your hearing system is a CIC or — IIC, insert by holding the hearing system with the removal handle at the



Gently insert the canal portion of the hearing system into your ear using your index finger to push back and tuck the hearing system completely into place. It should fit snugly and comfortably. Gently pull down and back on your earlobe to help you insert the hearing system into your ear canal

Removing your hearing system:

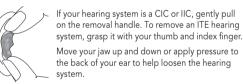

Note: Do not use the controls or the battery door to remove your hearing systems.

Turning Your Hearing Systems on and off.

Your hearing systems have a battery door that acts as an on/off switch and that allows access to the battery compartment.



On: Close the battery door fully.

Note: It may take 5 seconds before the hearing system turns on. Your hearing healthcare provider can increase the start up delay if




further adjust them – a rotary dial and a push button. IIC hearing systems may also come with a IIC remote.

With Rotary Dial

Adjusting Volume.

If the rotary dial has been configured as a volume

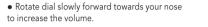

• Rotate the dial slowly backward away from your nose to decrease the volume.

Your hearing systems may come with two controls that allow you to



If the pushbutton has been configured as a volume

- Push the button on your right hearing system to increase the volume in both hearing systems.
- Push the button on your left hearing system to decrease the volume in both hearing systems.

With Remote Control

Refer to the remote control user guide.

Volume setting	Beep	s
Suggested volume level	1	1 beep
Turning volume up	1	short beep
Turning volume down	1	short beep
Maximum volume level	11	2 beeps
Minimum volume level	11	2 beeps

Switching Hearing Programs. With Pushbutton

• Use the program pushbutton to select one of the hearing programs

- which program you have selected.

Program 1 (e.g. AutoSurround)	ſ	1 beep
Program 2 (e.g. speech in noise)	11	2 beeps
Program 3 (e.g. PhoneConnect)	111	3 beeps
Program 4 (e.g. music)	ונונ	4 beeps
PhoneConnect	111	short melody
Wireless synchronization	Opposite	e ear beeps the same

as side being adjusted

With Remote Control

(if enabled)

• Refer to the remote control user guide.

Hearing Program/Description

1		
2		
3		

Tinnitus Manager

If your hearing healthcare professional has configured both a Tinnitus Manager program and a volume control, you can adjust the Tinnitus Manager level while you are in the Tinnitus Manager program.

• If wireless synchronization is enabled, then changing the volume or individually programmed for you. program setting on one hearing system, will automatically make the change to both hearing systems at the same time. Wireless synchroni-• When changing, your hearing system uses signal sounds to indicate zation works for both the push button and the lever. Please refer all questions concerning your hearing system to your • You can thus adjust your system to the relevant hearing situation. hearing system professional.

Binaural Phone (BiPhone)

Wireless Synchronisation (BiLink)

The Bi feature enables you to hold a phone to one ear and hear the sound clearly in both ears. No accessories are required.

Power-on-Delay

Optional Functionalities.

• allows you to turn the hearing systems on with an automatic delay of 5, 10, or 15 seconds to avoid feedback (whistling).

Telephone Coil

The telephone coil allows inductive signal reception (e.g., when telephoning). The reception of noise through the microphone is reduced or suppressed.



Distributed in USA by

Hansaton Acoustics Inc. 13845 Industrial Park Blvd Plymouth MN, 55441

www.hansaton-usa.com order@hansaton.com

Manufactured by

Hansaton Akustik GmbH Sachsenkamp 5 20097 Hamburg Germany

www.hansaton.com

uDirect 2/3, uStream, uTV3

The uDirect 2/3, uStream and uTV3 are optional accessories for your hearing systems. For more information on using the uDirect 2/3, uStream and uTV3, please see the user guide for that accessory. For information on how to get a uDirect 2/3, uStream or uTV3 accessory, please contact your hearing healthcare professional.

Automatic Telephone Program

 Hold a telephone near your hearing system: the hearing system automatically switches to the telephone program. Move the telephone away from your hearing system: the hearing system switches back to the previously used program.

Not all telephones produce a magnetic field strong enough to activate the PhoneConnect function. In this case, use the PhoneConnect magnet, enclosed in your hearing system package.

Attach it to the telephone receiver to activate the PhoneConnect

When operating the hearing system in telecoil mode, the wireless function may cause a pulsing noise. This is due to the pick-up of the transmission signal.

• When your hearing system is in telecoil mode please use the remote control only with a distance greater than 10 cm (4 inches).

Accessories.

Remote Control RCV1/2

The wireless function of your hearing systems allows you to use a remote control to change the program and volume of your hearing systems.

For detailed information and instructions, refer to the user guide, or ask your hearing system professional.

PhoneConnect

Your hearing system professional may have configured a telephone program, which you can reach via the automatic program switching function (PhoneConnect). The magnet on the telephone (optional) pro vides the magnetic field required to activate the telephone program.

Your hearing system professional can put together your own individual user manual for you.

IIC Remote

The IIC remote contains a magnet that functions like a remote control to change the settings of your hearing system. Your IIC remote can be configured as:

Program switch

Volume control (increase or decrease the volume)

To use the IIC remote, keychain housing unscrew the magnet from the keychain housing and hold it inside the ear canal without touching the hearing system.

Please refer all questions concerning your hearing system to your hearing system professional.

Advice on Care.

You are advised to use our HADEO care products to clean your hearing system. These products were developed specifically for this purpose and are available exclusively from your hearing system

- The hearing system itself must **not** be placed in water or cleaning
- The hearing system must be kept dry. Please put it in a HADEO DryBox, HADEO Drying Bag or Drying Container overnight to extract any moisture from the inside of the system.
- Clean your hearing system (preferably in the morning) with a cleaning tissue, keeping it
- Residues can be removed from the sound outlets with a small brush (allow system to dry completely first). Please do not use sharp objects to clean the system.

Care Primer

Our HADEO care primer can be obtained from your hearing system professional who will be glad to advise you on any questions you may have about caring for your system.



Troubleshooting.

Should a problem occur, you can usually correct it quickly. Use the following troubleshooting tips to resolve simple issues. If you still encounter problems after trying these suggestions, consult your hearing system professional.

Possible Cause

Hearing systems not

inserted properly.

Hand/clothing near

Hearing systems

Telephone not posi-

tioned properly.

Hearing systems

Low battery.

Low volume.

Weak on the

telephone.

Remove and reinsert

Move hand/clothing

away from your ear.

system professional.

system professional.

Consult your hearing

Replace battery.

Turn up volume.

Move telephone receiver

around ear for clearer

Consult your hearing

Poorly fitting hearing Consult your hearing

Poorly fitting hearing Consult your hearing

blocked with ear wax. system professional.

requires adjustment. system professional.

Problem	Possible Cause	Possible Solution
No sound.	Not turned on. Low/dead battery.	Turn on. Replace battery.
	Poor battery contact.	Consult your hearing system professional.
	Battery upside down.	Insert battery plus (+) side up.
	Hearing systems blocked with ear wax.	Consult your hearing system professional.
Not loud enough .	Low volume.	Turn up volume; see hea- ring system professional for models without a manual volume control or if problem persists.
	Low battery. Hearing systems not	Replace battery. Remove and reinsert
	inserted properly. Change in hearing.	carefully. Consult your hearing
	change in nearing.	system professional.
	Hearing systems blocked with ear wax.	Consult your hearing system professional.
Intermittent.	Low battery. Dirty battery contact.	Replace battery. Consult your hearing system professional.













jamHD jam flow

Instructions For Use

See also "General Warnings and Safety Instructions", Page 11-20





